



- Check test strip table; clean if needed.
- Run controls. (See package insert.)
- If **ERROR** occurs, press gray option key beneath **HELP** for printed explanation; press key beneath **CONT** to repeat test. (You may need to turn Analyzer *off* and back *on*.)

CLINITEK® *Microalbumin* Test Strips

- No special training required.
- Refer to bottle label for Reagent Strip storage and expiration date.




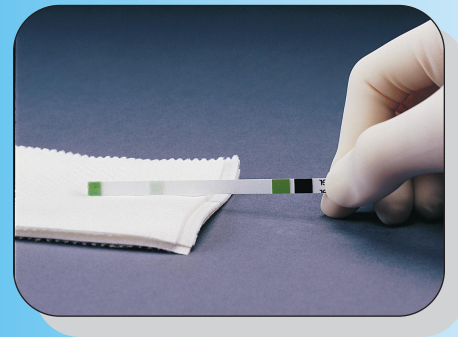
- 1. Testing** can be started only from READY FOR TEST screen.
- 2. Remove** a Reagent Strip from the bottle and replace cap.



- 3. Dip** a Reagent Strip into urine sample, wetting both pads. Do not dip colored bands near handle. **Immediately** remove strip.



- 4. Immediately press** either green  key **as you remove** strip from urine. Drag strip edge against side of container as you remove it. You now have 10 seconds to complete Steps 5 and 6.



- 5. Blot** by touching the **edge** of strip to paper towel to remove excess urine.



- 6. Place** Reagent Strip in middle trough of test strip table, with test pads facing up. Slide strip to **end** of trough. Do not push or pull table; it is automatically pulled into the Analyzer. Results are obtained in one minute.



- 7. Discard** strip when test is finished. Gently wipe table with a damp tissue, if needed.
- 8. Report** results to lab supervisor or physician.